



DeRoyal has created the ideal secondary dressing with our COVADERM® family of wound products. Save time and money by replacing gauze, tape, and scissors with our innovative non-adherent wound dressings.

With rounded corners and more shapes and sizes than ever, COVADERM®, COVADERM® THIN, COVADERM **PLUS**®, & **NOW** COVADERM **PLUS**® THIN are here to support you in creating an ideal wound healing environment.



COVADERM® adhesive wound dressing



- Replace gauze, tape, and scissors with an easier solution
- Conforms securely to body contours
- Breathable layer prevents drainage strike through and is a barrier to external contaminants
- Comfortable rounded edges and stretchable fabric tape
- Protective, absorbent sterile pad
- Non-irritating adhesive tape
- Split release liner for easy one-step application

COVADERM® THIN adhesive wound dressing



- Non-adherent, general purpose absorbent sterile dressing
- Conforms easily to wound
- Bulk case quantities - no individual boxes to open
- Comfortable rounded edges and stretchable fabric tape border
- Split release liner for easy one-step application

Product #	Overall	Pad	Qty/Bx	Bx/Cs	Qty/Cs	HCPC
46-001	4" x 4"	2½" x 2½"	25	8	200	A6254
46-002	4" x 6"	2½" x 4"	25	8	200	A6254
46-003	4" x 8"	2" x 5½"	25	4	100	A6254
46-004	4" x 10"	2" x 7½"	25	4	100	A6254
46-005	4" x 14"	2" x 11"	25	4	100	A6255

indications:

- Pressure ulcers
- Leg ulcers
- Chronic secreting lesions
- I.V. sites
- Laparotomy sites
- Arthroscopic incisions
- Tube sites
- Diabetic ulcers
- Surgical incisions



Product #	Overall	Pad	Qty/Cs	HCPC
15-201	4" x 4"	2½" x 2½"	100	A6219
15-202	4" x 6"	2½" x 4"	100	A6219
15-203	4" x 8"	2½" x 5½"	50	A6219
15-204	4" x 10"	2½" x 7½"	50	A6219
15-205	4" x 14"	2½" x 11"	50	A6220

indications:

- Pressure ulcers
- Leg ulcers
- Chronic secreting lesions
- I.V. sites
- Laparotomy sites
- Arthroscopic incisions
- Tube sites
- Diabetic ulcers
- Surgical incisions





PLUS V.A.D.



COVADERM PLUS® composite wound dressing



- Thick multi-layered composite wound dressing
- Fabric tape conforms to difficult/odd body contours
- Helps to maintain moist environment with protection from outside contaminants
- Breathable layer prevents drainage strike through and is a barrier to external contaminants
- Four layer construction:
 - Protective non-adherent contact layer
 - Soft pad for drainage absorption
 - Breathable, semi-occlusive polyurethane barrier film
 - Conformable adhesive-coated fabric tape
- Available with radial slit for tube sites

Product #	Size	Pad	V-Tape	Qty/Bx	Qty/Cs	HCPC
46-405	4" x 4" V.A.D.	2" x 2"	1" x 4"	25	200	
46-406	6" x 6" V.A.D.	4" x 4"	1" x 6"	5	50	

Product #	Tape	Pad	Qty/Bx	Qty/Cs	HCPC
46-400	2" x 2"	1" x 1"	25	100	A6203
46-411	2½" x 2½"	1½" x 1½"	10	50	A6203
46-401	4" x 4"	2½" x 2½"	10	50	A6203
46-402	6" x 6"	4" x 4"	10	50	A6203
46-403	6" x 8"	4" x 6"	10	50	A6204
46-414	4" x 10"	2" x 7½"	N/A	100	A6203
46-415	4" x 14"	2" x 11"	N/A	100	A6204
46-408	4" Diameterw/2" Radial Slit	2½" Diameter	25	200	A6203

indications:

- Pressure ulcers
- Leg ulcers
- Chronic secreting lesions
- I.V. sites
- Laparotomy sites
- Arthroscopic incisions
- Tube sites
- Diabetic ulcers
- Surgical incisions



COVADERM PLUS® THIN composite wound dressing



- NEW thinner multi-layered composite wound dressing
- Fabric tape conforms to difficult/odd body contours
- Non-adherent pad with not adhere to wound bed or staples/sutures
- Absorbent pad helps to control and maintain drainage
- Rounded corners help to prevent dressing from lifting off prematurely
- Four layer construction is ideal for a variety of wound types and sizes
- Bulk case quantities - no individual boxes to open
- Helps to prevent drainage strike-through
- Helps to prevent wound trauma

Product #	Size	Qty/Cs	HCPC
15-301	2x2	50/cs	A6203
15-302	4x4	50/cs	A6203
15-303	6x6	50/cs	A6203
15-304	6x8	50/cs	A6204
15-305	4x10	50/cs	A6203
15-306	4x14	50/cs	A6204

indications:

- Pressure ulcers
- Leg ulcers
- Chronic secreting lesions
- I.V. sites
- Laparotomy sites
- Arthroscopic incisions
- Tube sites
- Diabetic ulcers
- Surgical incisions



Improving Care. Improving Business.®



Additional Sizes/
Customization Available



Sterile



Burn Safe



Antimicrobial
Barrier



moist wound

application

1. Peel back edge of folded release paper.
2. Anchor exposed edge of tape onto skin, then carefully peel off rest of release paper.
3. Smooth all tape edges onto skin.



additional steps for V.A.D.

4. Peel back edge of release paper on V-cut strip.
5. Place tape under catheter as shown.



application types

1. CHRONIC WOUNDS:

Covaderm Plus is ideally sized for most chronic wounds from pressure sores and leg ulcers to other secreting lesions.

2. I.V. SITES:

Use Covaderm Plus to cover a discontinued I.V. Site.

3. SURGICAL WOUNDS:

Use Covaderm THIN for dressing laparoscopy sites after surgery. Use the larger Covaderm sizes for incisions that cover a broad surface area. Such as incisions associated with c-sections, saphenous vein harvest, open heart surgery, etc.

4. ORTHOPEDICS:

The 2" x 2" Covaderm is perfect for covering incisions made during arthroscopic knee surgery. Use the 4" x 4" to cover incisions made during ankle surgery and to provide a secure, comfortable covering over joint pins under orthopedic casts.

5. TUBE SITES:

Covaderm Plus is also available in a circular configuration with a radial slit for use around tube sites.



DeRoyal®, DeRoyal logo, Improving Care. Improving Business.®, and Covaderm® are registered trademarks of DeRoyal Industries, Inc.

*All study and clinical data claims are on file.