DEROYAL® RECORDABLE VOICE MONITOR

	NON-STERILE
	MEDICAL DEVICE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

i[ⓘ] Please read all instructions, warnings and precautions before use. Correct set up is essential for proper functioning of product.

This product is to be used by a licensed caregiver only for the use it was intended.

INTENDED USE

The DeRoyal® Recordable Voice Monitor is intended to alert staff and patients in the event the patient attempts to exit the bed, wheelchair and/or recliner.

⚠️WARNINGS

Failure to comply with these warnings may result in injury or death. This device is not suitable for all individuals. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive patient mobility management program. Test this device before each use. Do not use without reading the instructions.

This monitor is to be installed by a licensed caregiver. As such, it is the entire responsibility of the caregiver to ensure that the system is not designed to replace good caregiving practices including but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for fall prevention and/or elopement
- Testing of the system before each use

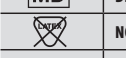
Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

SETTING UP THE MONITOR AND SENSOR PAD

- Open battery compartment door with a small Phillips-head screwdriver. Insert batteries.
- Close the battery door and secure it with the small Phillips-head screwdriver.
- Place one bed sensor pad or chair sensor pad within reach.
- Connect the monitor to the bed sensor pad or the chair sensor pad. Plug the cord end of the sensor pad into the bottom of the monitor. Pinch the cord together and route it through the cord protection recess on the monitor (near the plug input).
- Put pressure on the sensor pad to activate the pad and fall monitor. The fall monitor will beep two times to indicate that the sensor pad and fall monitor are in use. The "Reset/Status"

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button will blink while pressure remains on the pad. When pressure is removed, the fall monitor will sound the alarm. Press the "Reset/Status" button to silence the alarm.

- If the "Pad Lost Light" is on, check to make sure the cord is plugged in and reset the system. The ideal distance between the monitor and sensor pad is 15 to 20 feet.
- Test the system before use.

CUSTOMIZE A RECORDING

- Slide the "Select" switch to the "Record" (Rec) position. Press and hold the "Record/Play" button. While the red light on the back of the monitor is on, record your message and release the "Record/Play" button. Next, slide the "Select" switch to the "Play" position. Press the "Rec/Play" button to play back the message. **NOTE:** If you wish to record again, slide to "Rec" position and repeat the above process. Leave the slide switch in the "Play" position when recording is complete. **NOTE:** If using the customized recording option, the pre-recorded melody will no longer be available.
- To adjust the volume, turn the volume dial located inside the battery compartment.

MOUNTING THE MONITOR

- The strap with buckle can be used to attach the monitor to a desired location.
- To mount the monitor, mount the screw onto the wall or headboard. Hang the monitor on the screw using the mounting hole on back of the monitor.

TURNING OFF OR RESETTING THE ALARM

- When the alarm is triggered, the alarm can be turned off and reset with either the reset button or Cargiver Key™ function.
- Select either "Reset Button ON" or "Caregiver Key™ ON" with the switch on the side of the monitor. Use the end of a pen to move the switch.
- If set on "Reset Button ON" position, the caregiver can reset the monitor by pressing the reset button located on the front of the monitor.
- If set on the "Caregiver Key ON" position, the monitor can only be reset by using the Caregiver Key provided with the monitor. The Caregiver Key must be placed over the red dot to reset the monitor.

OPTIONAL ITEMS

- AC Adaptor can be purchased separately.
- Nurse call port can be purchased separately if the patient's room does not supply one that accommodates this device.

SIZING

Universal

CARE INSTRUCTIONS

Wipe clean with disinfectant wipes. After cleaning, test device before each use.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal

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Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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