




DEROYAL® TEMPERATURE THERAPY WRAP

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

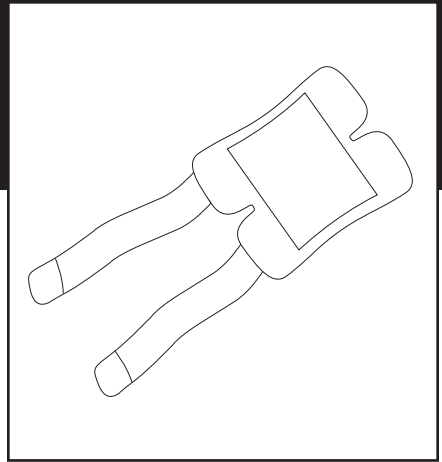

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INDICATIONS

COLD THERAPY: Use of cold therapy is intended to be prescribed by a properly licensed practitioner to temporarily reduce pain or soft tissue swelling after trauma or surgery.

CONTRAINDICATIONS


- **DO NOT** use cold therapy on patients with any of the following contraindications: Sickle-cell Anemia, circulatory syndromes due to Raynaud's disease, vasospastic disorder, cold allergy, cold hypersensitivity, previous cold injury, or any compromised local circulation, decreased skin sensitivity, hypertension, paroxysmal cold hemoglobinuria, cryoglobulinemia, and diabetes.

WARNINGS FOR HEALTHCARE PROVIDERS AND PATIENTS

- Monitor the cold gel pack temperature and condition of skin when in use.
- **DO NOT** sit, sleep, or lean against a cold gel pack to apply extra pressure.
- **DO NOT** use cold gel packs directly on the skin.
- Always place cold gel pack in the provided black pocket before use.
- Never use the product on an open wound.
- **DO NOT** use the cold gel pack if damaged.
- Dispose in accordance with local, state, and federal regulations.
- Exercise extra care when applying to the elderly.
- Consult your doctor before use.
- **DO NOT** use on infants.
- If you experience pain, swelling, allergies, or an unusual reaction while using this product, discontinue use and consult your healthcare professional immediately.


INSTRUCTIONS FOR USE

1. Place the cold gel pack in the freezer at 0° F (-18° C) and freeze for at least 2 hours.
2. After freezing, place the cold gel pack into the black pocket and tuck under the pocket flap. Secure the pocket closed with hook/loop (See Figure 1).
3. Place the black pocket on the inside of the wrap in the area cold therapy is needed and secure with hook/loop (See Figure 2).

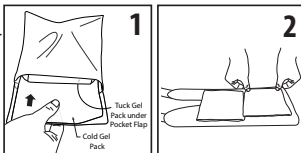


KNEE

1. Place the wrap on the knee with the longer strap (marked with DeRoyal logo) on the thigh.
2. Secure wrap to the leg by wrapping the thigh and calf strap around the leg.

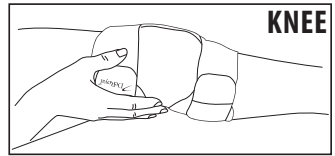


HIP/BACK

1. Place the wrap on the hip/back and secure it to the waist by overlapping the straps.
2. A 24" thigh strap can be used to secure the wrap to the leg.
3. A 24" extension strap can be used to extend the waist or thigh strap.

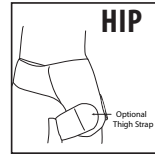


SHOULDER

1. Secure the arm and torso strap to the back of the wrap.
2. Place the wrap on the affected shoulder.
3. Wrap the torso strap around the body and secure to the front of the shoulder wrap. To tighten the torso strap, unfold the excess strap, pull to tighten, and reattach.
4. Wrap the arm strap under the arm and secure to the wrap.
5. To remove/reapply, detach/attach the torso strap with the easy strap release tab.

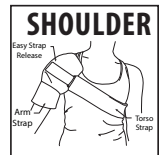


ARM

1. Place the wrap on the arm.
2. Secure the wrap to the arm by wrapping the straps around the arm.

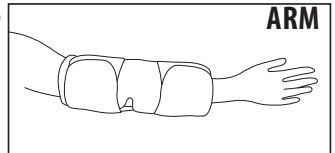




UNIVERSAL MEDIUM – GEL PACK PLACEMENT

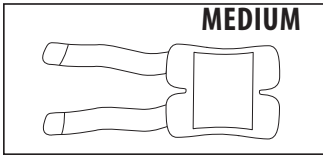

UNIVERSAL LARGE – GEL PACK PLACEMENT

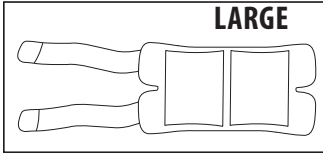

CARE AND MAINTENANCE

Rinse the product with mild detergent and air dry. Do not dry in a dryer or use other heat sources. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, towel dry or air dry (do not machine dry). Do not use softeners or bleach.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from deroyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured for:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#0-1901 | REVISED 12/2024

©2024 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are registered trademarks of DeRoyal Industries, Inc.