



JETOX™-ND
jet lavage wound cleansing & debridement system


For thorough yet virtually painless cleansing and debridement of wounds. Jetox™-ND gently and efficiently treats diabetic ulcers, pressure ulcers, venous stasis ulcers and burns. Administers a cool, gentle jet stream that creates a desensitizing effect for the patient. The treatment of choice for patients sensitive to other forms of debridement. Does not require suction or power source.



wand



tubing set

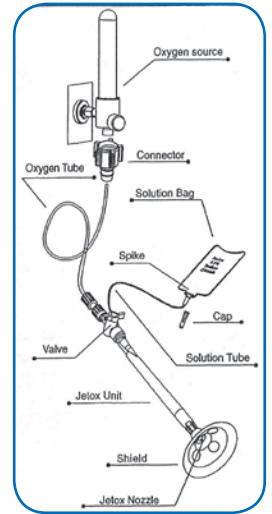

connection schematic

JETOX™-ND

jet lavage wound cleansing & debridement system

- Initiates thorough cleansing using compressed air/oxygen combined with a minimal amount (1.5ml/min) of standard sterile solution
- Quickly and efficiently cleans and debrides wounds
- Administers a cool, gentle jet stream that creates a desensitizing effect
- The treatment of choice for patients sensitive to other forms of debridement
- Uses standard connectors for portable and wall gas outlets and saline dispensers, does not require a vacuum system, generator or gun
- Ideal for bedside application in hospital, clinic or home settings
- Completely disposable, with virtually no cleanup of the work environment
- Easy set-up, lightweight and portable
- Easily controlled PSI
- Microdrops between 5-200 microns accelerated up to 200 meters per second

indications:

- | | |
|----------------------|-----------------|
| Diabetic ulcers | Burns |
| Pressure ulcers | Abrasions |
| Venous stasis ulcers | Trauma injuries |

PSI CHART

Open the oxygen source to appropriate setting for desired PSI.

- 9 L/Min = 4 PSI
- 11 L/Min = 6 PSI
- 13 L/Min = 9 PSI
- 15 L/Min = 12 PSI

Scan QR Code to view video links:



Product #	Description	Qty/Cs
48-500	Jetox-ND Wand	25
48-501	Jetox-ND Tubing Set	25
48-444	Jetox-ND Mini-Pack w/5 Wands, 5 Fit All Tubing Sets	5
48-555	Jetox-ND Mini-Packw/5 Wands, 5 Tubing Sets	5
48-601	Jetox-ND Fit All Tubing Sets	25



- | | | | | | | | | |
|---------------------------|-----------------------------|------------------------------------|-------------|--------------|--------------------------------|-------------------|-----------|-----------------------|
| 1 Primary Dressing | 2 Secondary Dressing | Not made with Natural Rubber Latex | Rx Required | Non-Staining | Additional Sizes/Customization | Available Sterile | Burn Safe | Antimicrobial Barrier |
|---------------------------|-----------------------------|------------------------------------|-------------|--------------|--------------------------------|-------------------|-----------|-----------------------|

DeRoyal® and DeRoyal logo, and Improving Care. Improving Business.® are registered trademarks of DeRoyal Industries, Inc.

Jetox™-ND is a trademark licensed for use by DeRoyal Industries, Inc.

Copyright© 2016, DeRoyal. All rights reserved.