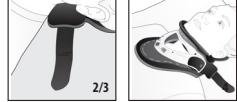
PREPARATION / PREPARACIÓN / PREPARAÇÃO / PRÉPARATION

APPLICATION / APLICACIÓN / APLICAÇÃO / MISE EN PLACE

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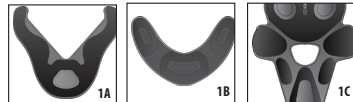
Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#0-1937
Revised 8/2024
©2024 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, and Capital are trademarks or registered trademarks of DeRoyal Industries, Inc.

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FR: COLLIER CERVICAL AVANCÉ CAPITAL™

Take special care to avoid contaminating hook parts with lint or debris.

REAPPLYING THE COLLAR PADS

- To reapply the collar pads to the collar frame, ensure you have all three collar pads: **CHEST PAD**, **CHIN PAD**, and **BACK PAD**.
- To reapply the **CHEST PAD** to the collar chest frame, fold the chest pad in half, evenly, and place the folded edge directly in between the two (2) hook dots on the front of the inner side of the chest frame. Attach the chest pad accordingly to the hook dots across the bottom and top corners of the chest frame. **NOTE:** The corners of the chest pad should only just cover the circular rivet at each corner of chest frame. Ensure the top crest of the chest pad is evenly aligned with the trachea opening of the chest frame. Ensure there is ample and even padding coverage across the entire chest frame.
- To reapply the **CHIN PAD** to the collar chin frame, fold the chin pad in half, evenly, and place the folded edge directly in between the two (2) hook dots on the inner side of the chin frame. Attach the chin pad accordingly to the hook dots on the chin frame and over to the top corner to meet the chest padding in each corner. **NOTE:** The corners of the chin pad should overlap the corners of the chest pad. Ensure there is ample and even padding coverage across the entire chin frame.
- To reapply the **BACK PAD** to the collar back panel, align the opening of the back pad with the opening of the back panel. Attach the back pad accordingly to all the hook dots across the bottom and top corners of the back panel. Ensure there is ample and even padding coverage across the entire back panel.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYALS WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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squeeze both adjustment buttons on the sternal frame of the collar, push down on the chin portion of the collar, then release the adjustment buttons. If buttons are not able to be compressed, the center circular adjustment lock may need to be rotated counter clockwise to the unlocked position, then adjustment can be made.

APPLICATION

- NOTE:** Ensure all open wounds are covered before applying the device.
- To apply, the patient's head should be in **NEUTRAL ALIGNMENT**.
 - Slide the back portion of the collar behind patient's head and neck. Ensure panel is centered and there is equal loop strap length on both sides of the patient's head and neck.
 - The loop straps should be located between the trapezius and the patient's ear.
 - Slide the front panel of the collar up the patient's neck and position the sides of the panel over the patient's trapezius. Align the bottom of the tracheal opening at the sternal notch. **NOTE:** It is okay if the patient's chin is not touching the chin piece.

SIZING ADJUSTMENT

- While gently holding the front panel against the patient's chest, squeeze both adjustment buttons on the sternal frame of the collar. Hold the center of the chin portion, lift it until the chin piece is touching the patient's chin, and release the adjustment buttons on the sternal frame of the collar. Height indicating notches will be visible on either side of the chin support.
- Once the chin piece is adjusted to the desired height, the adjustment buttons can be locked via the circular adjustment lock located in the center of the sternal frame of the collar. To lock the adjustment buttons, turn the circular adjustment lock ¼ turn "clockwise" or to where the center line of the circular adjustment is horizontal. The lock indicating arrows will be aligned. To unlock, turn the circular adjustment back up ¼ turn "counterclockwise" to where the center line of the circular adjustment is vertical.
- If the height of the chin piece needs to be readjusted, unlock the circular adjustment lock, squeeze both adjustment buttons on the sternal frame of the collar, push down on the chin portion of the collar, then release the adjustment buttons.

FITTING

- With the front portion of the collar properly adjusted, use loop straps and fasten the front of the collar to the back panel of the collar via the hook on the left and right sides of the front portion of the collar.
- NOTE:** Ensure the strap is applied precisely and parallel to the hook material on the front collar frame. This will ensure proper alignment between the front and back panels of the collar.
- Minor adjustments can be made to the straps to achieve desired comfort and a secure, symmetrical fit.
- For more information on correct application and fitting of the Capital™ Advanced Cervical Collar, go to www.deroyal.com.

CARE INSTRUCTIONS

- The collar features three removable pads that can be cleaned with mild soap and water. Squeeze out excess water and air dry flat.
- Replacement pads are available for use during washing.
- If needed, collar plastic parts can be wiped using mild soap and a damp cloth.

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CAPITAL™ ADVANCED CERVICAL COLLAR

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Capital™ Advanced Cervical Collar is intended to restrict cervical flexion, extension, lateral bending, and axial rotation to promote patient recovery.

WARNINGS

- A physician or properly licensed practitioner, (a "prescriber") who is familiar with the purpose for which the brace is intended must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the neck (3) a limitation on duration of use, or (4) some combination of these.
- Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.
- Inspect brace for damaged or missing components prior to patient application.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged. Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Monitor the skin for any reactions, irritation, redness, or swelling.

INSTRUCTIONS FOR USE

PREPARATION
Ensure the front panel is at the lowest height setting. If adjustment is necessary,

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