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Manufacturer:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or 865.938.7828
www.derooyal.com

EC REP: ONET B.V.
Kansstraat 19
NL-5076 NP Haaren
The Netherlands

Part #0-2220G
Revised 8/2016
©2017 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.

Boa is a registered trademark of Boa Technology, Inc.

DEROYAL® SPORTS ORTHOSIS
POWERED BY THE BOA® SYSTEM

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Sports Orthosis is for chronic instabilities of the ankle, prophylactic talocrural support and prevention and treatment of acute ankle sprains.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a wearing period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the ankle (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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SIZING

SIZE	MEASUREMENT*
X-Small	10"-11"
Small	11"-12"
Medium	12"-13"
Large	13"-14"
X-Large	14"-15"
XX-Large	15"-16"

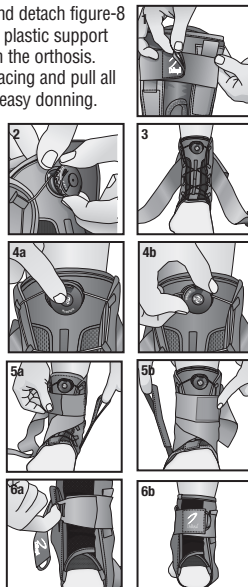
* Measure circumference from the base of the heel around the ankle.

INSTRUCTIONS FOR USE

NOTE: A sock should be worn under this brace.

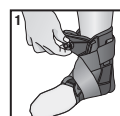
APPLICATION

- Open rear closure straps and detach figure-8 straps. **NOTE:** If applicable, plastic support stays can be removed from the orthosis.
- Pull out on dial to release lacing and pull all lace out of dial to assist in easy donning.
- Place foot in brace at a 90° angle.
- Push dial in to engage then turn knob clockwise to tighten laces. Laces should be secure but comfortable.
- Starting with inside (medial) strap, bring strap across the top of the foot. Wrap strap under the heel and back around attaching to side of the brace using hook and loop closure. Repeat in the opposite direction for outside (lateral) strap.
- Secure rear closure straps.
- Readjust as needed for a secure and comfortable fit.
- Laces will equalize across the top of the foot with movement. Adjust as needed during activity.

REMOVAL

- Pull out on dial to release lacing and pull all lace out of dial for ease of removal.
- Open rear closure and unattach figure-8 straps.
- Slide foot out of orthosis. **NOTE:** It is recommended to assemble all

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straps and lacing when orthosis is not in use to prevent tangling and excess wear.

CARE INSTRUCTIONS

If applicable, remove plastic stays from medial and lateral pockets, hand wash softgoods in lukewarm water, mild detergent, air dry.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
--	----------

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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