



Improving Care. Improving Business.®

TOILET PAD

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and precautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

This device is intended to alert staff and patients in the event the patient attempts to get up from the toilet without assistance.

WARNINGS

- The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program.
- Discontinue use of device and consult physician or properly licensed practitioner if it become damaged in any way, limiting its ability to perform as intended.
- Inspect product for damaged or missing components prior to application.
- **FAILURE TO COMPLY WITH WARNING MAY RESULT IN INJURY OR DEATH.** This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. Sensor pad and fall monitor systems are prone to difficulties. Be sure that the cord cannot entangle the user. This device is designed to augment caregivers' comprehensive resident mobility management program. Test this device before each use. Read the instructions and Legal Disclaimer before each use.

INSTRUCTIONS FOR USE

1. Install new batteries into the DeRoyal® fall monitor. Required battery type is indicated inside the battery compartment. The battery compartment door is secured with a small Phillips-head screw.
2. Plug the cord end of the sensor pad into the bottom of the DeRoyal® fall monitor. Pinch the cord together and loop it through the strain relief recess on the fall monitor (near plug input).
3. Test the system. Put pressure on the sensor



pad to activate the pad and fall monitor. The fall monitor will beep two times to indicate that the sensor pad and fall monitor are in use. The "In Use" or "Status" light will blink when pressure is on the pad. When pressure is removed, the fall monitor will sound the alert tone. Press the reset button to silence the alert .

4. **Suggested fall alarm mounting:** Mount on the wall above the closed toilet tank.
5. Sensor pad must be placed on top of the feet on the underside of the toilet seat to operate properly.
6. If suggested mounting location is used, run the cord of the sensor pad under and behind the toilet tank as illustrated above.
7. Connect the sensor cord to the fall monitor. Be sure that the cord cannot entangle or choke the user.
8. While the patient sits on the toilet, the system is in use. When the patient gets up from the toilet, the alert will sound.

CLEANING AND MAINTENANCE

- For safety, replace this pad after 30 days of use.
- When using a sensor pad in conjunction with foam or gel cushions, test the system to make sure it functions properly.
- A new sensor pad must be used of each patient.
- To clean sensor pads, use disinfectant wipes or anti-bacterial cleaners.
- **DO NOT** fold the sensor pad.
- **DO NOT** immerse the sensor pad in any liquid cleaning solution.
- **DO NOT** expose the sensor pad to high temperature sanitizers.

TROUBLESHOOTING

- Make sure that the sensor pad is plugged into the fall monitor.
- Check the power supply. If you are using battery power, install new batteries. Or, if you are using an AC adapter, make sure that the adapter is plugged into a power outlet.
- Check monitor switches: A. When using a monitor with a Safety Auto-Reset™ button: For normal operation (in Reset mode) the switch on the left-hand side of the fall monitor must be in "Reset Button On" position. To silence the fall monitor, press the "Reset" button or place pressure on the



Improving Care. Improving Business.®

sensor pad (to automatically reset the monitor).B. If you are using a fall monitor with an On/Off switch (on left side of the monitor), make sure the switch is in the “On” position.

LEGAL DISCLAIMER

The DeRoyal® Sensor Pad and Fall Monitor System is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to ensure that the system is properly installed and tested. Also, DeRoyal expressly disclaims any and all liability when third party sensor pads are used with DeRoyal fall alarm hardware. DeRoyal has not tested other manufacturers' sensor pads with DeRoyal's fall monitor hardware and; therefore, strongly cautions against connecting a DeRoyal fall alarm to another manufacturers' sensor pads. The use of another manufacturers' sensor may damage the alarm, cause the fall monitoring system to not function as intended, and will void any remaining DeRoyal warranty. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- a.) Direct patient supervision
- b.) Adequate training for staff personnel for resident fall prevention and elopement
- c.) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



Improving Care. Improving Business.®



Manufactured For:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#0-2320 Revised 2/2020

©2020 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.
Safety Auto Reset is a trademark of Smart Caregiver, Inc.