

Omni-Jugs®

for collection & aspiration of
large volumes of medical / surgical fluids

How to Assemble

Step 1:

Align lid with handles and place on the jug.

Step 2:

Press firmly to snap all eight lock-tabs around entire perimeter of lid.

Step 3:

Connect vacuum port on the jug to suction source on wall or ceiling.

Step 4:

Connect suction tubing from field to appropriate ports on the jug's port cap.

Step 5:

Place jug in standard ring stand or floor dolly.

Step 6:

Seal unused ports.



Improving Care. Improving Business.

DeRoyal Industries, Inc.

200 DeBusk Lane | Powell, TN 37849 USA
888.938.7828 or 865.938.7828 | www.deroyal.com