



DeRoyal has created the ideal secondary dressing with our COVADERM® family of wound products. Save time and money by replacing gauze, tape, and scissors with our innovative non-adherent wound dressings.

With rounded corners and more shapes and sizes than ever, COVADERM®, COVADERM **PLUS**®, & **NOW** COVADERM® AG with an antimicrobial barrier layer are here to support you in creating an ideal wound healing environment.



# COVADERM® AG

antimicrobial composite wound dressing



- Provides quick and sustained antimicrobial barrier against a broad spectrum of external wound pathogens including methicillin-resistant Staphylococcus aureus (MRSA), Staphylococcus aureus, Pseudomonas aeruginosa, E. Coli \*
- Helps to maintain moist environment with protection from outside contaminants
- Multi layer construction:
  - Protective non-adherent antimicrobial contact layer
  - Soft pad for drainage absorption
  - Conformable adhesive-coated fabric tape
- Easy application and removal
- Ideal for many wound types

## COVADERM® AG

antimicrobial composite wound dressing



Conformable adhesive-coated fabric tape

Soft pad for drainage absorption

Protective non-adherent antimicrobial contact layer



Product #	Tape	Pad	Qty/Bx	Qty/Cs	HCPC
46-0044AG	4" x 4"	2½" x 2½"	25	200	A6203
46-0046AG	4" x 6"	2½" x 4"	25	200	A6203
46-0066AG	6" x 6"	4" x 4"	25	100	A6204
46-0048AG	4" x 8"	2½" x 5½"	25	100	A6204
46-0410AG	4" x 10"	2" x 7½"	25	100	A6204
46-0414AG	4" x 14"	2" x 11"	25	100	A6204
<b>VAD</b>					
46-0044AGV	4" x 10"	-	25	200	-
46-0066AGV	4" x 14"	-	25	100	-

### indications:

- |                           |  |
|---------------------------|--|
| Pressure ulcers           | Surgical incisions                         |
| Leg ulcers                | Donor sites                                |
| Chronic secreting lesions | More severe lacerations, cuts, & abrasions |
| I.V. sites                | Superficial wounds                         |
| Tube sites                |  |

### contraindications:

- |                         |                                |
|-------------------------|--------------------------------|
| Third degree burns      | Patient sensitive to alginates |
| Heavily bleeding wounds |                                |



## application

### Primary dressing application

1. Peel back edge of folded release paper.
2. Anchor exposed edge of tape onto skin, then carefully peel off rest of release paper.
3. Smooth all tape edges onto skin.

### VAD application

1. Peel back edge of release paper on V-cut strip.
2. Place tape under catheter as shown.



DeRoyal®, DeRoyal logo, Improving Care. Improving Business.®, and Covaderm® are registered trademarks of DeRoyal Industries, Inc.

\*All study and clinical data claims are on file.

Copyright© 2018, DeRoyal. All rights reserved.