


Sensor Application

TEMPLE TOUCH PRO™
Non-Invasive Core Temperature System

1

Wipe Temple Area Clean


- » Use alcohol wipe
- » Allow to dry or wipe dry with gauze

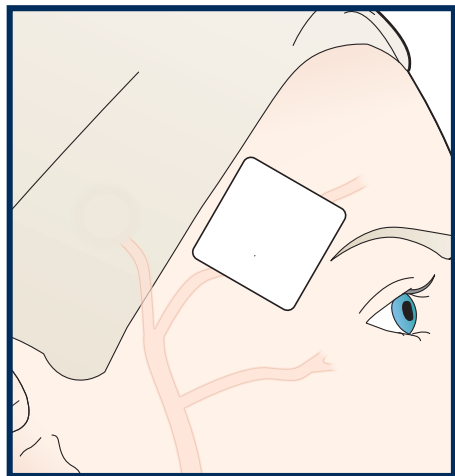

2

Connect Sensor to Sensor Connecting Unit (SCU)


- » Hold close to connection points when connecting
- » Carefully insert the sensor tab
- » Confirm blue stripes align

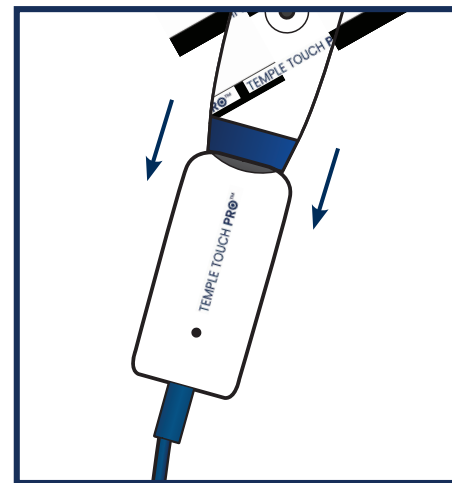

3

Remove Adhesive Backing & Apply Sensor over Temporal Artery


- » Ideal placement location is between the orbital ridge and hairline

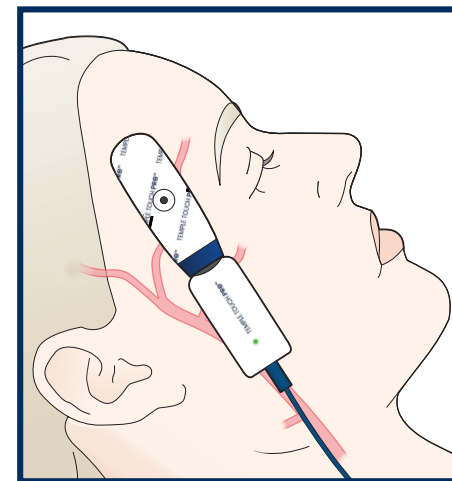

4

Attach SCU Securement Clip

- » Prevent unintended drag/pull on sensor
- » Use gown, bouffant cap, pillow, sheet, etc.

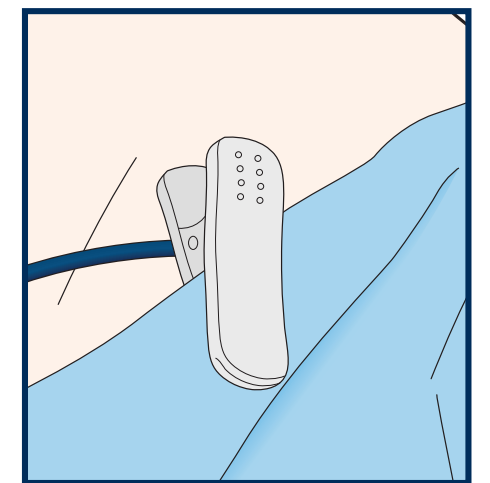

⚠ CAUTION

- » Do not bend sensor tab
- » Do not place sensor tab over hair
- » Limit use of sensor to 24 hours
- » Avoid exposing sensor to surgical prep solutions or the other fluids
- » Do not reposition sensor as it can impact system performance

Provided for information only. Not for diagnostic use.

DeRoyal Industries, Inc. | 200 Debusk Lane | Powell, TN, 37849 | 888.938.7828 | DeRoyal.com
The DeRoyal logo is a registered trademark of DeRoyal Industries, Inc. The Temple Touch Pro logo is a trademark of Medisim, Inc.

© 2025, DeRoyal. All Rights Reserved. | Rev. 2/25 | LIT0-2632

DeRoyal®