


1

Wipe Temple Area Clean



- » Use alcohol wipe
- » Allow to dry or wipe dry with gauze

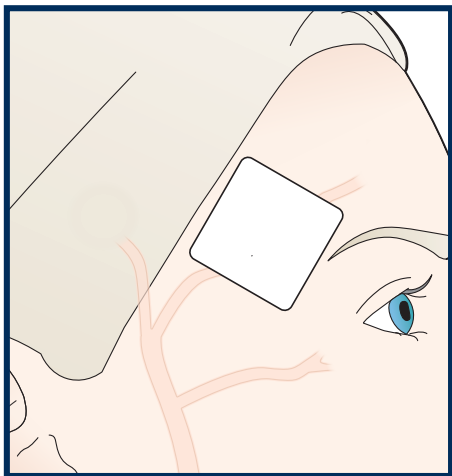

2

Remove Adhesive Backing & Apply Sensor over Temporal Artery

- » Place sensor on temporal artery
- » Avoid placing sensor on “soft spot”, hairline, or sinuses
- » Gently press for 2-3 seconds
- » For **ENT procedures**, place sensor on non-operative side
- » For **BIS™ monitoring**, apply TTP sensor prior to applying BIS sensor

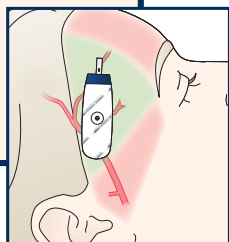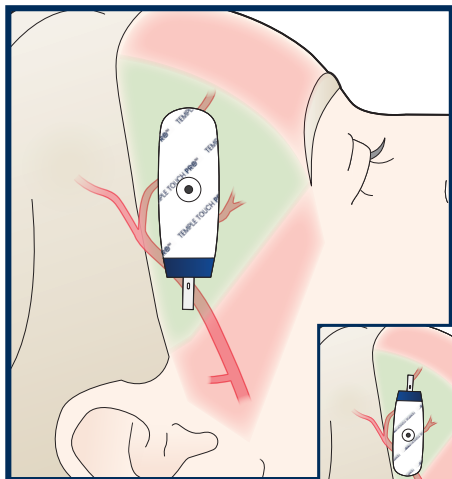

IMPORTANT ⚠


- » Do not bend sensor tab
- » Limit use of sensor to 24 hours
- » Avoid exposing sensor to surgical prep solutions or other fluids
- » Do not reposition sensor as it can impact system performance

- » For **prone procedures**, place sensor so that tab is toward the top of the head

3

Connect Sensor to Sensor Connecting Unit (SCU)

- » Hold close to connection points when connecting
- » Carefully insert the sensor tab
- » Confirm blue stripes align

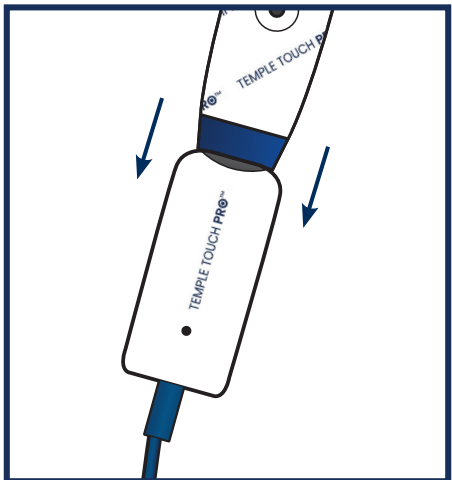

4

Attach SCU Securement Clip

- » Prevents unintended drag/pull on sensor
- » Use gown, bouffant cap, pillow, sheet, etc.

