



Mount standard rail clamps to the side rails of the torso/back portion of the surgical table. Insert the flat portion of the securement bars into the standard rail clamps, with the cylinder portion of the bar extending across the OR table. For ease of adjustment, do not tighten the clamps at this step.



Place the positioning pillow on the OR table and insert the securement bars fully into the channels on the back of the pillow. If included, place the lift sheet across OR table. Do not place directly over the top of the ramp. If needed based on the patient positioning, the lift sheet can be placed underneath the ramp.



Transfer the patient onto the OR table. Move the pillow up or down the table to reposition as needed. Ensure patient is positioned with head in center cutout and that there is no more than a hands-width (0.25 in | 0.75 cm) gap between pillow and patient's shoulder. Once patient is properly positioned, secure the product to the OR table by fully tightening the rail clamps to the securement bars.

If included, secure the patient's arms using the lift sheet or foam arm protectors.



Place the chest strap across the patient and secure to the bed railing tautly.



OR



Before procedure begins, do an assessment to ensure the patient is securely positioned, and that the surgical team is in agreement with the final patient positioning, or as per facility protocol.