## PRONE PATIENT POSITIONING KIT

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## PRONE PATIENT POSITIONING KIT

	SINGLE PATIENT USE
	NON-STERILE
	DO NOT USE IF PACKAGE IS DAMAGED
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

### IMPORTANT

Please read all instructions, warning, and cautions before use. Correct application is essential for proper functioning of the product.

### INTENDED USE

The DeRoyal® Prone Patient Positioning kit is intended to provide support, stability and comfort for patient protection when placed in a prone position

### WARNINGS

- Prior to use, inspect the kit components for damaged, missing, or contaminated parts.
- Use only as directed.
- Follow all facility protocol for patient turning and monitoring. Regularly inspect patient for signs of injury due to prolonged positioning. Replace foam kit with a new kit when necessary.

### CAUTIONS

The components within this system have only been tested in conjunction with each other. Use of outside components could alter proper performance.

### DIRECTIONS FOR USE

- Properly position patient flat on abdomen in the center of the bed.
- With patient's head and neck in optimum alignment, place face down or to one side of the headrest.
- Gently move the patient's arms and hands placing them parallel to the body or raise one arm up with the elbow flexed and other arm parallel to the body (swimmers' position), then place one standard size pad under each elbow. **NOTE:** if patient is in swimmers' position their head should be turned toward the raised limb.
- Place the large size pad under the patient's knees.
- Place patient's feet in the anatomically correct position maintaining flexion by elevating with the shin positioners.

## STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.