



Improving Care. Improving Business.®

SECURITY CUFF WITH QUICK RELEASE BUCKLE

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was prescribed to by a healthcare professional and only for the use it was intended.

INTENDED USE


The DeRoyal® Security Cuffs with Quick Release Buckle are intended to be prescribed by a properly licensed practitioner to protect the patient from harming themselves, other patients, family members and/or staff.

CONTRAINDICATIONS

Never use on a patient if an I.V. or wound site could be compromised by the device.

! WARNINGS

- Keep straps out of patient's reach. Patient should not be able to remove device. Improper application or use of any restraint may result in serious injury or death.
- Inspect product for damage, missing components, or contamination before use. Ensure that all hook and loop and buckle closures function properly.
- Use hospital approved bed safety guidelines. If securing the device to a bed or chair, always secure straps to the portion of the bed or chair that moves with the patient. Do not secure straps to bed side rails. Always keep straps out of patient's reach.
- Check the patient regularly per hospital's policies and procedures to ensure circulation is not compromised.
- After device is placed on the patient, ensure that all three buttons of the QR buckle are engaged. Incorrect insertion of the buckle may allow a patient to unintentionally exit the device.

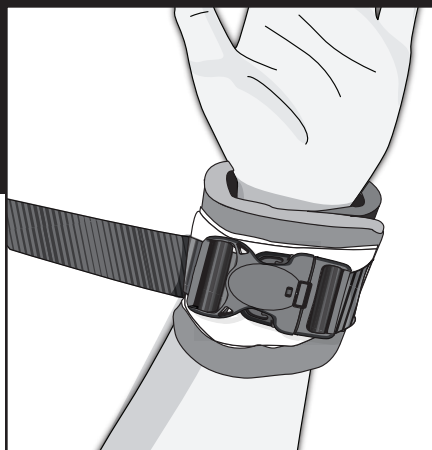


CAUTIONS


- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and cautions to other healthcare practitioners or healthcare providers involved in the patient's care.
- Consult physician (or properly licensed practitioner) immediately if patient experiences sensation changes, unusual reactions, swelling or increased pain while using this device. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE


1. To secure the attachment strap to the bed or gurney, release the female end of the buckle and wrap its strap around the anchor. Pull the female portion through the looped portion of the strap.
2. Connect the male and female ends of the buckle such that the white sticker completes a square, ensuring that all three points of the buckle are engaged. Pull the excess strap to tighten.

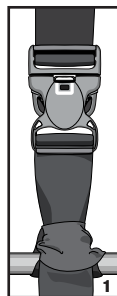

NOTE: For additional security, you can tie excess strap in a knot behind buckle. Secure excess strap away from patient's reach.

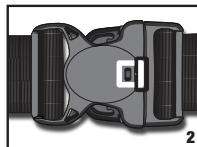





3. To secure the cuffs on the patient, open the cuff by releasing the quick release buckle and hook and loop, utilizing the small

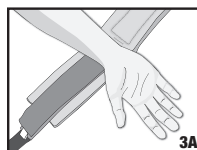



Improving Care. Improving Business.®

white tabs. Wrap the cuff around the patient's wrist or ankle with the soft foam side touching the patient's skin, then follow with the loop side securing it to the hook. Attach the other hook tab to the loop.

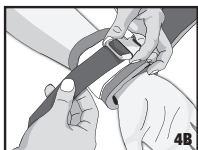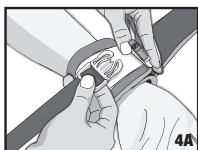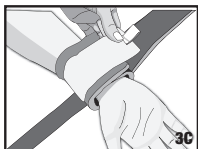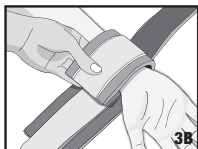

- 4. To secure the quick release buckle around the cuff, insert the male end into the female end of the buckle. Pull excess strap to tighten and secure the hook tab to the foam cuff ensuring hook does not touch patient's skin.

NOTE: The QR buckle will only release when applying pressure on all three buttons in Figure 2.

CLEANING AND/OR MAINTENANCE (IF APPLICABLE)

Please refer to hospital approved guidelines for cleaning and disinfecting of the device.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#0-4215 Revised 6/2022

©2022 DeRoyal Industries, Inc.
All Rights Reserved.

DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.