VOLAR WRIST SUPPORT

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SUPPORT DE POIGNET PALMAIRE

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VOLAIRE POLSSTEUN

VOLART HANDLEDSSTÖD

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VOLAR WRIST SUPPORT

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Volar Wrist Support is intended to support the wrist in a neutral position and is indicated for Carpal Tunnel Syndrome, tendonitis, wrist sprains and fractures.

CONTRAINDICATIONS

Not to be used if any of the following conditions exist: severe or fixed contractures; excessive edema or swelling; excessive pain.

CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand and wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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INSTRUCTIONS FOR USE

- To adjust wrist position, place support on volar surface of hand/forearm and assess fit of wrist position. Bend aluminum stay to conform to desired wrist position.
- To adjust middle strap, wrap middle strap across top of wrist to hook material on volar surface support. Strap should be positioned at distal end of hook material patch. Trim excess strap.
- To adjust distal strap, wrap distal strap so that it crosses over the dorsal side of the hand. Secure strap to hook material on volar surface of support, proximal to middle strap. Trim excess strap.
- To adjust proximal strap, bring proximal strap across forearm to attach to hook material patch on volar surface of support. Trim excess strap.

CARE INSTRUCTIONS

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.