## PUCCI® AIR-T INFLATABLE HAND ORTHOSIS

## ORTESIS DE MANO INFLABLE PUCCI® AIR-T

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## PUCCI® AIR-T INFLATABLE HAND ORTHOSIS

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

**IMPORTANT INFORMATION**  
Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

**INTENDED USE**  
The DeRoyal® Pucci® Air-T Inflation Hand Orthosis is intended to provide continuous low stretch therapy to the contracted hand.

**CONTRAINDICATIONS**  
Severe or fixed contractures; excessive edema or swelling; excessive pain.

### CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a wearing period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand and wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

### INSTRUCTIONS FOR USE

- Determine the degree of wrist position. If the wrist position is neutral, or 0° to 15° of extension, apply the Pucci® Air-T as it comes from the package. If the flexion contracture is 5° or more, bend the orthosis at the narrow portion, below the bladder, to conform to the wrist.

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- Insert the **BLACK END** of the inflation/deflation bulb into the air valve on the orthosis, and squeeze the bulb to deflate the bladder.
- Open all hook and loop closures on the orthosis. Position the air valve at the ulnar side of the hand and carefully slip the deflated bladder under the fingers as far as comfort will allow.
- Gently position the thumb around the bladder and into the positioning "groove." Fasten the hook and loop closures straps over the wrist and forearm. Check that the straps are secure and allow adequate circulation.
- Inflate the bladder per physician's order by inserting the **WHITE END** of the inflation/deflation bulb into the air valve on the orthosis and squeezing.
- After inflation, fasten the hook and loop closure straps over the hand or fingers to maintain the bladder in the correct position under the fingers and thumb.
- Recheck the fingers and thumb for desired position over the inflated bladder. Assess fingers for adequate circulation. If circulation is impaired, remove orthosis immediately.
- To remove the orthosis, deflate the bladder, and open all hook and loop closure straps.

**REPLACING COVERS**  
To remove the cover, deflate the bladder. Slide the frame out of the cover through the opening on the back. To replace the cover, insert the deflated frame into the clean cover, lining up the inflation valve with the small opening in the cover.

**CARE INSTRUCTIONS**  
**BLADDER, INFLATION BULB AND WRIST SUPPORT**  
Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

**COVER**  
Remove the liner and fasten all hook and loop closures. Place in mesh laundry bag. Machine wash on gentle cycle with mild detergent and warm water. **DO NOT** use bleach or fabric softener. Rinse in cold water and air dry. Covers can be washed but not sterilized.

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

**WARRANTY**  
DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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