**PUCCI® REFLEX INHIBITORY POSITION (R.I.P.) HAND ORTHOSIS**

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**PUCCI® REFLEX INHIBITORY POSITION (R.I.P.) HAND ORTHOSIS**

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

**IMPORTANT INFORMATION**

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

**INTENDED USE**

The DeRoyal® Pucci® Reflex Inhibitory Position (R.I.P.) Hand Orthosis is intended to provide continuous therapy to the contracted fingers, hand and wrist.

**CONTRAINDICATIONS**

Severe or fixed contractures; excessive edema or swelling; excessive pain.

**CAUTIONS**

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a wearing period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand and wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

**INSTRUCTIONS FOR USE**

- Determine the angle of the wrist position. Form the orthosis to conform to the wrist. **NOTE:** The R.I.P. is shipped with the finger platform attached. The finger platform attaches by

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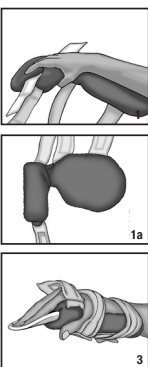
sliding through the opening of the cover and into the slit of the orthosis. Remove when only hand roll is needed (1a).

- Open hook and loop closures. Carefully open the fingers and position them on the R.I.P. finger platform, with the wrist and forearm against the forearm section of the orthosis.
- Fasten the hook and loop closures over the forearm, wrist and fingers. Check that they are secure and allow adequate circulation.
- Check the fit of the entire orthosis, and adjust, if necessary, to ensure proper fit. The wrist portion can be bent to fit an increased range of motion position.
- When the R.I.P. finger platform is no longer needed, it can be removed, and the hand roll can be added. Insert the loop strap of the orthosis through the opening in the hand roll. Slide the hand roll onto the palmar section of the orthosis.
- Once the orthosis is fitted, check the fit. Follow your facility's protocol. If no protocol is provided by your facility, check the fit of the orthosis at increasingly longer intervals of time. Start with the orthosis on for 15 minutes and off for 15 minutes. **DO NOT** exceed seven hours on at any time.
- Recheck the fingers and thumb for desired position over the hand roll. Assess fingers for adequate circulation. If circulation is impaired, remove orthosis immediately.

**REPLACING COVERS**

To remove, slide the cover off the forearm support and palmar section. Slide the cover off the finger platform or the hand roll. Reapply on the platform for the main frame of the splint by sliding the cover over the forearm support and palmar section of the splint. If using the hand roll, slide the hand roll cover of the splint over the hand roll frame.

**CARE INSTRUCTIONS**

**FRAME** - Clean the frame with a mild detergent and water.

**COVER**

Remove the cover and fasten all hook and loop closures. Place in mesh laundry bag. Machine wash on gentle cycle with mild detergent and warm water. **DO NOT** use bleach or fabric softener. Rinse in cold water and air dry prior to wearing. Covers can be washed but not sterilized.

**STORAGE AND TRANSPORT CONDITIONS**

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

**WARRANTY**

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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