
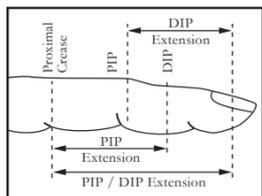
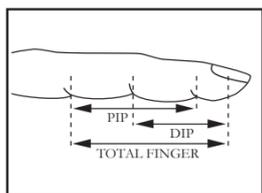
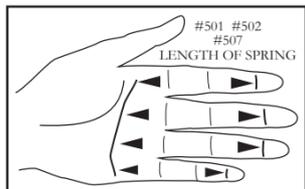
## SIZING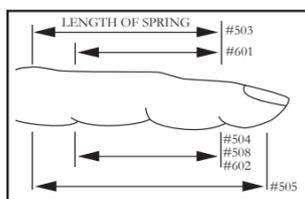

## FINGER SPRING PROXIMAL INTERPHALANGEAL EXTENSION

SINGLE PATIENT USE	
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any product.

### INTENDED USE

The DeRoyal® Finger Spring fits from the proximal phalanx to the distal interphalangeal (DIP) joint. It is used to assist the proximal interphalangeal joint (PIP) into increased extension. It does not interfere with the metacarpophalangeal (MP) joint.

### CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the product to build muscle strength (2) regular physical therapy exercises to maintain the strength of the finger (3) a limitation on duration of use or (4) some combination of these.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the product or its components break or become damaged.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

## SIZING

To determine splint length, measure distance from MP crease to distal finger crease on volar surfaces as indicated in the diagram.

SIZE	LENGTH	
AA	1 3/4"	4.4 cm
A	1 7/8"	4.7 cm
B	2"	5.1 cm
C	2 1/4"	5.7 cm
D	2 5/8"	6.6 cm

### INSTRUCTIONS FOR USE

#### FITTING THE SPRING (A)

- Hold the spring at the proximal and distal pads. **NOTE:** The proximal pad has the white plastic covered wire attached to it.
- Slide splint onto finger so the proximal pad rests over the volar surface of the proximal phalanx and the distal pad rests at the distal interphalangeal joint (DIP).

#### ADJUST SPRING TENSION (B)

- To increase the force of extension, spread the volar pads apart before placing on the finger.
- To decrease the force of extension, pinch the volar pads together before placing on the finger.

### CARE INSTRUCTIONS

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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