## WRIST SUPPORT

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## SUPPORT DE POIGNET

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## POLSSTEUN

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## EL BILEGI DESTEĞI

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## WRIST SUPPORT

SINGLE PATIENT USE	
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

### INTENDED USE

The DeRoyal® Wrist Support uses wire coils at the wrist joint axis to assist the wrist and digits into extension. The thumb outrigger assists the thumb into abduction/extension. This splint is recommended in injuries involving the radial nerve.

### CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand and wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

### SIZING

Splint size is determined by the width of the hand at the metacarpophalangeal joints.

SIZE	WIDTH
B	2 ¼" - 2 ¾" (5,71 - 6,98cm)
C	2 7/8" - 3 ¼" (7,28 - 8,26cm)
D	3 ¾" - 3 ¾" (8,57 - 9,53cm)

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




### INSTRUCTIONS FOR USE

- To fit the splint, loosen all strapping from the radial side. Slide hand into splint so the contoured palm support rests in the palmar arch and the distal bar rests along the volar surface of the proximal phalanx of the fingers.
- When strapping the elastic wrist strap, note that the strap remains fastened around the ulnar coil. Cross the strap over the wrist. Wrap the strap around the wire extending from the radial coil. Adhere the loop material to the hook material. Once fastened, the strap will rest along the dorsum of the wrist.
- On the wrist, attach the folded end of the strap to the radial side of the splint. Secure the splint at the wrist by pulling the strap through the ulnar slit and adhering it to the adjacent hook material. Securing the splint will position the ulnar support close to the wrist.
- To strap the forearm, wrap the strap around forearm so the hook material on the folded end of strap makes contact with strap along the volar surface of the splint. Trim excess strapping.
- To increase the wrist extension, bend the lateral wires into extension force. The bend should be equal in the radial and ulnar wire. Slide the plastic tubes up along the elastic attached to the finger outrigger. This brings the fingers into extension. Tie the elastic and cut.
- With the hand positioned in the splint, slide the thumb cuff on the thumb so it rests at the proximal phalanx. The wire attached to the thumb cuff can be adjusted for desired thumb extension/abduction.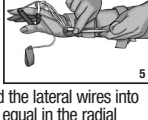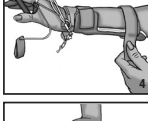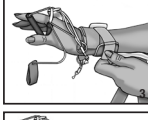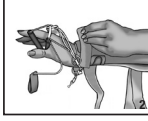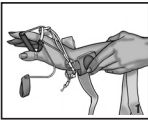

### CARE INSTRUCTIONS

Straps may be removed and laundered. Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

### STORAGE AND TRANSPORT CONDITIONS

KEEP DRY	
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.