



MULTIPAD™

	DO NOT REUSE
STERILE EO	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions and warnings before use.

GENERAL

Multipad is a multilayered, atraumatic speciality absorptive dressing composed of a thick rayon highly absorbant nonwoven pad and two polypropylene wound contact layers. Both sides are non-adherent and will not disturb granulation tissue or cause trauma to the wound site.

INTENDED USE

Multipad can be used as a primary or secondary dressing over Multidex®, Kalginate®, and Algidex Ag® dressings, as well as other wound fillers. Multipad can be used in place of gauze and tape and/or other non-adherent dressings to support more absorbancy; no wound contamination from fiber linting; less frequent dressing changes; no trauma to wound site on removal; or to save nursing time.


INDICATIONS

Multipad is used for:

- Dermal ulcers
- Diabetic ulcers
- Abdominal wounds
- Infected wounds
- Superficial wound
- Lacerations, cuts, and abrasions
- Donor sites
- Second degree burns

CONTRAINDICATIONS

- Not used for second degree burns
- Not for use on patients with a known allergy to the pad components (Rayon and Polypropylene)

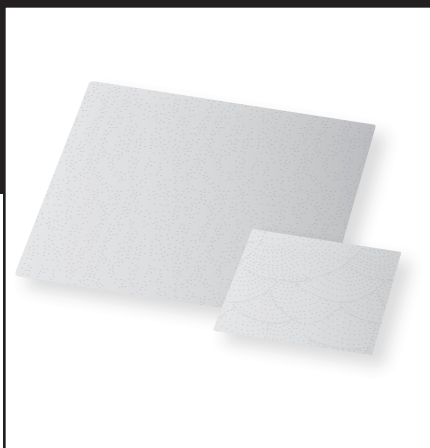

WARNINGS

- Observe wound for any erythema beyond dressing edges which may indicate a sensitivity to the dressing, wound infection or exacerbation of an already infected wound.
- A heavily soiled dressing or a dressing showing 'strike through' exudates should be removed and replaced to avoid wound and/or skin maceration.
- Local wound care cannot overcome the deficits of unrelieved malnutrition, pressure or trauma, or compromised blood flow
- Interventions must be instituted to reduce/relieve these factors that contribute to impaired wound healing in order to achieve the optimum outcome.

DIRECTIONS FOR USE

APPLICATION

1. Peel open package using aseptic technique.
2. Position Multipad over wound.
3. Secure with tape, roll gauze, or StretchNet™ Tubular Elastic Bandage.

REMOVAL

1. To remove Multipad gently lift dressing from wound; clean and redress wound.
2. Dispose of soiled dressing as a biohazard according to facility policy.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.



WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from deroyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#4-4444 | REVISED 2/2025

©2025 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal, Multipad, and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.