EN: TRANSEAL® TRANSPARENT FILM DRESSING

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PT: CURATIVO EM PELÍCULA TRANSPARENTE TRANSEAL®

FR: PANSEMENT FILM TRANSPARENT TRANSEAL®

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.derooyal.com

Part#4-6391 | Revised 3.2023
©2023 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Transeal are registered trademarks of DeRoyal Industries, Inc.

TRANSEAL® TRANSPARENT FILM DRESSING

	DO NOT REUSE
	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in injury.

GENERAL

Transeal® Transparent Film Dressing is a sterile, transparent polyurethane film coated with a medical grade adhesive. The Transeal® dressing aids in wound healing by creating a "second skin" that controls the environment in which the wound will heal. Transeal® dressing transmits moisture vapor, but is a barrier to water and external contaminants.

INTENDED USE

Transeal® Transparent Film Dressing is indicated for: Vascular Access Sites; Dermal Ulcers; 1st and 2nd Degree Burns; Graft and Donor Sites; and Surgical Incisions.

WARNINGS

- DO NOT** reuse, reprocess, or re-sterilize this dressing. This device is for single use only. Reuse, reprocessing, or re-sterilization may damage the dressing, and may result in injury or infection.
- DO NOT** use if the individual package is opened or if the package or the dressing is damaged. Carefully inspect the dressing for damage or defects prior to use.
- Seek medical attention if your wound is more severe than a minor abrasion, minor laceration, or minor scald and burn. Also seek medical attention if you have signs of an infection or allergic reaction such as fever, redness, pain, or swelling.
- If you have any signs of an allergic reaction, discontinue using the dressing and rinse the wound and surrounding skin.
- Ensure the dressing adheres to skin.
- If dressing is ineffective or does not adhere well to the skin, discard and replace.
- Dispose of all used dressings with care to avoid exposure to or contamination with biohazard.
- Ensure the dressing fully covers your wound.

CAUTIONS

- Ensure the carrier tab is removed after film application.
- If the dressing rips or tears prior to or during application, discard the torn dressing and replace it.
- Prior to removal, ensure the dressing has not adhered to the wound. If the dressing has adhered to the wound or skin, rinse with water or saline solution to loosen.

DIRECTIONS

- Clean and thoroughly dry site.
- Simultaneously peel release liner (Tab 1) from center of dressing while aligning the dressing over the site. Press the dressing into place.
- Remove the clear carrier film by lifting away at Tab 2.
- To remove Transeal®, gently grasp the edge and slowly pull the dressing from the skin in the direction of hair growth.
- Store at room temperature and avoid excessive heat and humidity.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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