**Explanation of Symbols****Explicación de los símbolos****Explication des symboles****Erläuterung der Symbole****Spiegazione dei simboli****Explicação dos Símbolos****Verklaring van de symbolen****Symbolförförklaringar****Sembollerin açıklaması****Επεξήγηση των συμβόλων**

Do not use if package is damaged  
No utilizar si el envase está dañado  
Ne pas utiliser si l'emballage est endommagé  
Bei beschädigter Verpackung nicht verwenden  
Non utilizzare se la confezione è danneggiata  
Não utilizar se a embalagem estiver danificada  
Niet gebruiken als de verpakking beschadigd is  
Använd inte om förpackningen är skadad  
Paket hasar ise kullanmayın  
Να μην χρησιμοποιείται αν η συσκευασία έχει υποστεί ζημιά

Improving Care. Improving Business.<sup>®</sup>**COVADERM<sup>®</sup> WOUND DRESSINGS****APÓSITOS PARA HERIDAS COVADERM<sup>®</sup>****PANSEMENTS DE PLAIES COVADERM<sup>®</sup>****COVADERM<sup>®</sup> WUNDVERBÄNDE****MEDICAZIONI PER FERITE COVADERM<sup>®</sup>****CURATIVOS PARA FERIDAS COVADERM<sup>®</sup>****COVADERM<sup>®</sup> WONDVERBANDEN****COVADERM<sup>®</sup>-SÅRFÖRBAND****COVADERM<sup>®</sup> YARA ÖRTÜLERİ****ΕΠΙΘΕΜΑΤΑ ΤΡΑΥΜΑΤΩΝ COVADERM<sup>®</sup>**

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Covaderm<sup>®</sup> wound dressings are sterile, absorbent, multi-layer composite wound dressings. They are available with and without a polyurethane barrier film layer. All Covaderm, Covaderm Thin, Covaderm Plus<sup>®</sup> and Covaderm Plus Thin have rounded edges, non-adherent contact layers, and easy to shape adhesive coated fabric tape.

**USE**

Covaderm wound dressings absorb wound drainage, promote a moist environment helpful to healing, and serve as a barrier against water and bacteria when used as a secondary dressing.

**INDICATIONS**

- Covaderm wound dressings are indicated for minor abrasions, minor lacerations, and minor scalds and burns.
- When under the care or direction of a licensed practitioner they are indicated for:
  - Dermal ulcers (e.g., pressure ulcers, leg ulcers, and other secondary lesions)
  - Second degree burns
  - Donor sites
  - More severe lacerations, cuts and abrasions
  - Other superficial wounds
  - Covaderm Plus VAD is also indicated for IV sites, and other tubular sites and incisions.

**CONTRAINDICATIONS**

- Covaderm wound dressings are contraindicated for:
  - Third-degree burns
  - Heavily bleeding wounds
  - Individuals who are sensitive to or have had an allergic reaction to the dressing or its materials

**WARNINGS**

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse, reprocess, or re-sterilize this dressing. This device is for single use only. Reuse, reprocessing, or re-sterilization may damage the dressing, and may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package or the dressing is damaged. Carefully inspect the dressing for damage or defects prior to use.
- Seek medical attention if your wound is more severe than a minor abrasion, minor laceration, or minor scald and burn. Also seek medical attention if you have signs of an infection or allergic reaction such as fever, redness, pain, or swelling.
- If you have any signs of an allergic reaction, discontinue using the dressing and rinse the wound and surrounding skin.
- If applicable, ensure the dressing adheres to skin. Dressings with weak adhesives should be replaced immediately.
- If dressing is ineffective or does not absorb, discard and replace.
- If applicable, dispose of all used dressings with care to avoid exposure to or contamination of biohazard.

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All Rights Reserved.