ADHESIVE DRESSINGS

1. Peel the paper liner from the back of the adhesive dressing.
2. Center the contact layer over the wound.
3. Anchor exposed edge of dressing adhesive onto skin, then carefully peel off rest of release paper.
4. Smooth all adhesive edges onto skin.

5. Change dressing as needed when wet, soiled, leaking, or as directed by a licensed practitioner.
6. To remove the dressing, hold the edge of the adhesive border and carefully pull it in the direction of hair growth, lifting bandage from the wound.
7. Dispose of the soiled dressing.

STORAGE AND TRANSPORT CONDITIONS

Store at room temperature and avoid excessive heat and humidity.

	KEEP AWAY FROM SUNLIGHT
	KEEP DRY

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

Manufacturer:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or 865.938.7828
www.deroyal.com

Part#4-7077

Revised 7/18

©2018 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business. and Covaderm are trademarks or registered trademarks of DeRoyal Industries, Inc.

COVADERM® AG COMPOSITE WOUND DRESSING

	DO NOT REUSE
	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

 Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or infection.

INTENDED USE


DeRoyal® Covaderm® Ag silver alginate wound dressings are intended to manage abrasions and lacerations, dermal wounds, donor sites, first- and second-degree burns, surgical incisions, vascular access sites, pressure ulcers stage I-IV, stasis ulcers, and venous ulcers.

CONTRAINDICATIONS

- Do not surgically implant.
- Do not use on:
 - Third degree burns.
 - Ulcers resulting from infections.
 - Lesions in patients with active vasculitis.
 - Patients with sensitivity to silver, silver compounds, or alginates.

WARNINGS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in injury.
- Do not reuse, reprocess, or re-sterilize this dressing. This device is for single use only. Reuse, reprocessing, or re-sterilization may damage the dressing, and may result in injury or infection.
- Do not use if the individual package is opened or if the package or the dressing is damaged. Carefully inspect the dressing for damage or defects prior to use.
- Seek medical attention if your wound is more severe than a minor abrasion, minor laceration, or minor scald and burn. Also seek medical attention if you have signs of an infection or allergic reaction such as fever, redness, pain, or swelling.

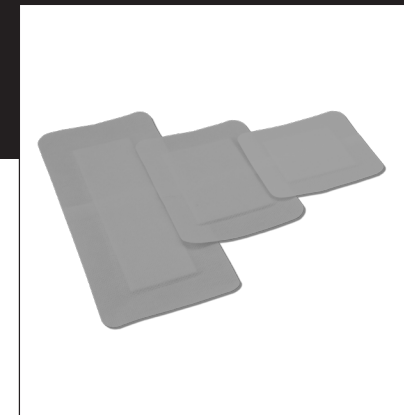- If you have any signs of an allergic reaction, discontinue using the dressing and rinse the wound and surrounding skin and contact the prescriber.
- Ensure the dressing adheres to skin.
- If dressing is ineffective or does not absorb, discard and replace.
- Dispose of all used dressings with care to avoid exposure to or contamination with biohazard.
- Ensure dressing components are not left in wound upon removal.

LOCAL WOUND CARE CAN NOT OVERCOME THE DEFICITS OF UNRELIEVED MALNUTRITION, PRESSURE, TRAUMA OR COMPROMISED BLOOD FLOW. INSTITUTE INTERVENTIONS TO REDUCE OR RELIEVE FACTORS THAT CONTRIBUTE TO IMPAIRED WOUND HEALING IN ORDER TO ACHIEVE THE OPTIMUM OUTCOMES.

CAUTIONS

- Ensure the dressing fully covers your wound.
- If the dressing rips or tears prior to or during application, discard the torn dressing and replace it.
- Prior to removal, ensure the dressing has not adhered to the wound. If the dressing has adhered to the wound or skin, rinse with water or saline solution to loosen.

DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Clean the wound of excess drainage and fluid, and thoroughly dry skin around the wound.
3. Open the dressing package.

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