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EN: UMBILICAL CORD CLAMP AND CLAMP CUTTER

	DO NOT REUSE
	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

The DeRoyal® Umbilical Cord Clamp is intended to occlude the blood vessels in the umbilical cord of a newborn infant as the cord dries and shrinks after birth.

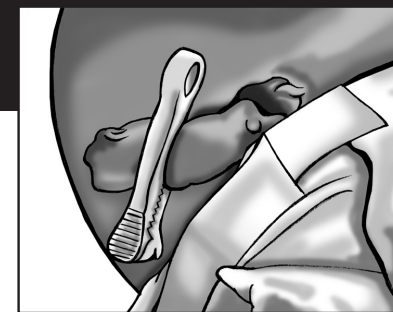
The DeRoyal® Umbilical Cord Clamp Cutter is intended to cut the umbilical cord clamp off the newborn.

WARNINGS

- This device must be applied by a healthcare professional who is familiar with its use. Improper use could cause severe injury.
 - For sterile product, **DO NOT** use if sterile packaging is damaged or opened. If damage is found, remove the product from the operative area to prevent unintended use.
- Take care in handling the device as it has sharp edges that could cause scratches, lacerations or glove perforations.
- Take care in handling the Cord Clamp Cutter as it has a sharp blade that could cause injury to the clinician.
- Dispose all items in accordance with facility protocol.

DIRECTIONS

- Visually inspect the clamp and clamp cutter for damage. Replace if device is damaged.
- Open packaging using proper sterile technique.
- Place the umbilical cord in the center of the clamp. Avoid placing the cord too close to the latch or hinge. Improper placement could result in injury to the patient.
- Press the cord clamp until it clicks, indicating the clamp is securely closed.
- Cut the umbilical cord in accordance with the current standard of care, but in no instance closer than ½ inch (13 mm) from the clamp. Cutting too close to the clamp could cause the clamp to slip off the cord.
- Remove the cord clamp in accordance with the current standard of care.
- If necessary to remove, cut the clamp at the hinge area with the clamp cutter until completely severed.

STERILIZATION INFORMATION

DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging and/or sterilization. These products are to be sterilized by ethylene oxide, utilizing the following parameters: Temperature: 120°F - 135°F (48.9°C – 57.2°C), Ethylene Oxide Gas Concentration: 625 – 800 mg/l, Exposure Time: 4 hrs minimum (Preconditioning: Temperature 100°F -120°F (37.8°C – 48.9°C), Humidity 55-75%, Time 18hrs minimum), (Aeration: Temperature 100°F -135°F (37.8°C – 42.2°C), Time 18hrs minimum). Other sterilization methods have not been validated by DeRoyal and may damage the product which could cause a device malfunction and/or injury to the patient.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#44-00009 | REVISED 6/2023

©2023 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.

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