



NEONATAL EYE COVER WITH TABS

	DO NOT REUSE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal[®] Neonatal Eye Cover with Tabs is intended to cover and protect the eyes of an infant during therapeutic procedures, such as phototherapy.

WARNINGS


- Ensure the eye cover does not obstruct the infant's breathing through the nose or mouth.
- Securely position the eye cover, ensuring the eye cover covers the infant's eyes completely.
- Ensure eye cover does not interfere with any monitoring equipment or treatment of infant.

PRECAUTIONS

- If skin irritation develops, discontinue use.
- This device is for single patient use only. It is not intended for disinfection and subsequent re-use, which may result in device failure and/or create the risk of contamination.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing or re-sterilization may damage the device, rendering it unusable or may lead to device failure, which could result in patient illness, injury or death.

DIRECTIONS FOR USE

1. Prep infant's skin with the alcohol pad provided.
2. Affix an acrylic tab to each side of the infant's head, between the ears and eyes.

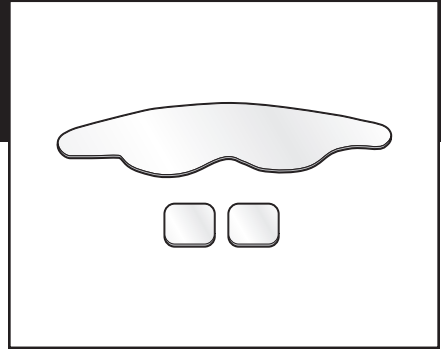

Secure firmly.

3. Center the eye cover over the bridge of the infant's nose, making sure both eyes are completely covered. The nose and mouth should be unobstructed.
4. If eye cover is removed temporarily, hook and loop tabs may be covered with cloth tabs.

STORAGE AND TRANSPORT CONDITIONS

DO NOT store product at extreme temperatures or in a moist/damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal[®] products are warranted for ninety (90) days from the date of shipment from deroyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART #44-00146 | REVISED 6/2023

©2023 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo
are trademarks or registered trademarks of DeRoyal Industries, Inc.