
DeRoyal uses the symbols and meanings from standards ISO 15523-1. These symbols are placed next to the text explaining their meaning in this instructions for use (IFU), a complete symbols glossary is available online at [deroyal.com/symbols](http://deroyal.com/symbols) or by contacting customer service.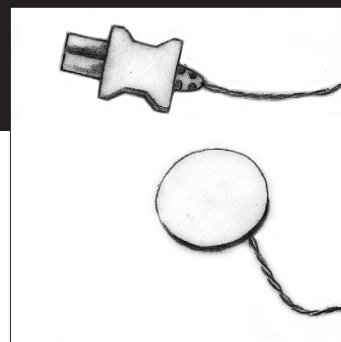

## EN: SKIN TEMPERATURE SENSOR

## ES: SENSOR DE TEMPERATURA CUTÁNEA

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## PT: SENSOR DE TEMPERATURA DA PELE

Manufactured by:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
[www.deroyal.com](http://www.deroyal.com)

### PART#74-14902

Revised 2/2025

©2025 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal and the DeRoyal logo are registered trademarks of DeRoyal Industries, Inc.

## SKIN TEMPERATURE SENSOR

|  |  |
|--|--|
|  | MR UNSAFE  |
|  | DO NOT RESTERTILIZE  |
|  | DO NOT REUSE   |
|  | STERILIZED WITH ETHYLENE OXIDE   |
|  | NOT MADE WITH NATURAL RUBBER LATEX   |
|  | FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER. |

### IMPORTANT

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury.

### INTENDED USE

The DeRoyal® Skin Temperature Sensor is to be used for routine monitoring of a patient's skin surface temperature.

### INTENDED USERS

The DeRoyal® Skin Temperature Sensor is intended to be used only by a licensed practitioner or trained healthcare professionals.

### PATIENT TARGET GROUP(S)

The intended patient population is pediatric and adults.

### PRODUCT DESCRIPTION

The Adult Skin Temperature Sensor consists of a temperature-sensing wire set with an adhesive sensor cover that facilitates application to the patient's skin surface. The temperature monitoring sensor located at the distal end of the wire is affixed to the center of the disk-shaped foam pad, which has a reflective backing and an adhesive layer to hold the sensor in place.

The temperature monitoring sensor is located at the tip of the sensor. The blue connector at the proximal end interfaces with DeRoyal-branded cables for connection to a monitoring unit to display temperature. The temperature monitoring sensor is available in YSI 400 or YSI

700 series configurations for connection to a compatible monitoring unit. Whether the sensor is a 400 or 700 series configuration is indicated on the device labeling. Consult the user manual for the patient monitor to ensure compatibility. The sensors are designed to interface with DeRoyal-branded cables for connection with YSI 400 or 700 series compatible monitors, including the following patient monitors and equivalent models: Mindray® Passport, Philips® IntelliVue®, Siemens/Dräger® Infinity®, and GE® Datex-Ohmeda® brands.

### INDICATIONS

The DeRoyal® Skin Temperature Sensor is indicated to monitor the patient's skin surface temperature.

### CONTRAINDICATIONS

None known.

### ADVERSE REACTIONS

While adverse reactions are rare, the following have been associated with use of the skin temperature sensor during use or immediately after use:

- electrical burns,
- skin irritation,
- inflammation, or
- allergic reaction to materials.

### ⚠️ WARNINGS

- The Skin Temperature Sensor should only be used on healthy adult skin.
- **DO NOT** apply the adhesive cover on this sensor to a skin surface which has been damaged by abrasion, laceration, incision, burning, irritation, chafing or inflammation.
- **DO NOT** use this skin sensor on neonatal patients.
- **DO NOT** use sensor without sensor cover. Use without sensor cover can cause inaccurate readings, and could potentially lead to patient injury.
- Carefully inspect the packaging and device for damage or defect. **DO NOT** use if the packaging is open or the device is damaged.

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### CAUTIONS

- Device is intended for use by a licensed practitioner.
- Take care to ensure the cable and connector do not get wet and proper techniques are used during electrosurgical procedures to reduce radio frequency interference, current and potential burns to the patient.
- An adequate electrosurgical dispersive ground electrode close to the active surgical site should be properly connected.
- **DO NOT** intertwine the cables, especially the monitor cables, with the electrosurgical unit's cables.
- The operation of the patient temperature monitor may be temporarily affected during electrosurgical activations. Unusual temperature readings should be checked.
- If temperature does not read properly, check all connections. If problem persists, discard and use a new sensor.
- **DO NOT** use an expired device.
- In case of adverse skin reaction, discontinue use.
- The skin sensor is intended to be used with DeRoyal-branded interface cables and sensor cover. Use with an incompatible cable may affect performance.

### ⚠️

### SAFETY IN MRI NOT EVALUATED.

The Adult Skin Sensor's safety in MRI has not been evaluated, hence, the sensor is considered MR Unsafe.

### DIRECTIONS FOR USE

1. Prepare the skin for adhesive pad by removing body oils and hair.
2. Verify the compatibility of the sensor, interface cable, and patient monitor before use.
3. Remove the Skin Temperature Sensor from its package.
4. Peel off release liner and apply sensor with sensor cover to patient's skin.
5. Connect the temperature sensor to the appropriate reusable interface cable from DeRoyal. Connect the cable to the temperature monitor. Secure the interface cable using the drape clip.
6. Follow the directions for use of the patient temperature monitor instrument.

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7. When temperature monitoring is completed, carefully remove adhesive cover from patient's skin.
8. Once temperature monitoring is no longer needed, disconnect the sensor (at the connector) and discard the sensor. To disconnect, grasp both connectors firmly and pull. **DO NOT** pull on cable or wire.

### OPERATING SPECIFICATIONS

- Operating Temperature: 25° C to 45° C
- Rated Output Range: 35° C to 42° C
- Rated Extended Output Range: 25° C to 35° C and 42° C to 45° C
- The reference body site is the skin surface temperature.
- The measuring site(s) are intact skin.
- Accuracy in Rated Output Range: +/- 0.2° C
- Accuracy in Rated Extended Output Range: +/- 0.3° C
- Time response:
  - The temperature sensor has a direct mode of operation.
  - The maximum heating transient time is 12 seconds
  - The maximum cooling transient time is 10 seconds

### DISPOSAL

Dispose of any used device according to local, state, and federal laws and regulations. For safe disposal of devices, follow your facility's protocol.

### STORAGE AND TRANSPORT CONDITIONS

|  |                         |
|--|-------------------------|
|  | KEEP DRY                |
|  | KEEP AWAY FROM SUNLIGHT |

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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