ALGIDEX AG® PACKING GAUZE

GASA DE ENVOLVER ALGIDEX AG®

GAZE DE PREENCHIMENTO ALGIDEX AG®

RUBAN DE GAZE ALGIDEX AG®

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART# 810880
Revised 6/2021

©2021 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business., and Algidex Ag are trademarks or registered trademarks of DeRoyal Industries, Inc.

ALGIDEX AG® PACKING GAUZE

	USE BY DATE
	DO NOT REUSE
	STERILIZED USING IRRADIATION
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

DEVICE DESCRIPTION

Algidex Ag® Packing Gauze is a sterile cotton gauze coated with an ionic silver alginate and maltodextrin matrix. Algidex Ag® Packing Gauze contains silver that is evenly distributed in the ionic state. The dressing provides a sustained antimicrobial activity in the dressing for up to three (3) days against a broad spectrum of microbial organisms.

INTENDED USE

Algidex Ag® silver alginate wound dressings are intended to manage exuding wounds, including abrasions and lacerations, dermal wounds, donor sites, first- and second-degree burns, surgical incisions, vascular access sites, pressure ulcers stage I-IV, stasis ulcers, and venous ulcers.

CONTRAINDICATIONS

- Algidex Ag® Packing Gauze should NOT be used:
 - on patients with sensitivity to silver, silver compounds or alginates
 - on third-degree burns

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- ulcers resulting from infections
- lesions in patients with active vasculitis
- DO NOT** surgically implant.

WARNINGS

- Local wound care cannot overcome the deficits of unrelieved malnutrition, pressure or trauma, or compromised blood flow. Interventions must be instituted to reduce or relieve factors that contribute to impaired wound healing to achieve the optimum outcome. When visible signs of infection are present, use these dressings in conjunction with proper systemic medical treatment to address the cause of the infection.
- DO NOT** use when contraindicated.
- Discontinue use if signs of sensitivity or irritation appear.
- Product is sterile unless packaging is opened or damaged. **DO NOT** use if product is expired or if packaging is open or damaged.
- Discard excess product after initial opening.
- DO NOT** use for longer than three (3) days.
- This product is prescription only. **DO NOT** provide it to the patient without providing complete use instructions.
- Remove product and cleanse wound thoroughly prior to hyperbaric oxygenation or any other therapy where the presence of metals is contraindicated.

CAUTIONS

- DO NOT** store in extreme temperatures or in a damp environment. This may render the product ineffective.

DIRECTIONS FOR USE

- Always wear gloves when dealing with open wounds.
- Thoroughly cleanse the wound with saline.
- Completely fill the wound with gauze, packing it lightly.
- Cover the wound with the appropriate secondary dressing.

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- Change as desired, at a minimum every three (3) days.
- To change, remove the product gently.
- Remove any product residue with saline, if desired.

EXPIRATION INFORMATION

- Store Algidex Ag® Packing Gauze in a cool, dry place.
- Contents sterile unless package has been opened or damaged.
- See package label for expiration date.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT
	DO NOT USE IF PACKAGE IS DAMAGED

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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